


PleurX™ Pleural Catheter System 50-7000B for pleural effusion management

ChlorPrep™ 10.5 mL Applicators with Hi-Lite Orange™ Tint (qty. 2) J-tip guidewire, 60 cm Guidewire introducer with needle Metal tunneler Dilator, 12 Fr Catheter insertion styllet

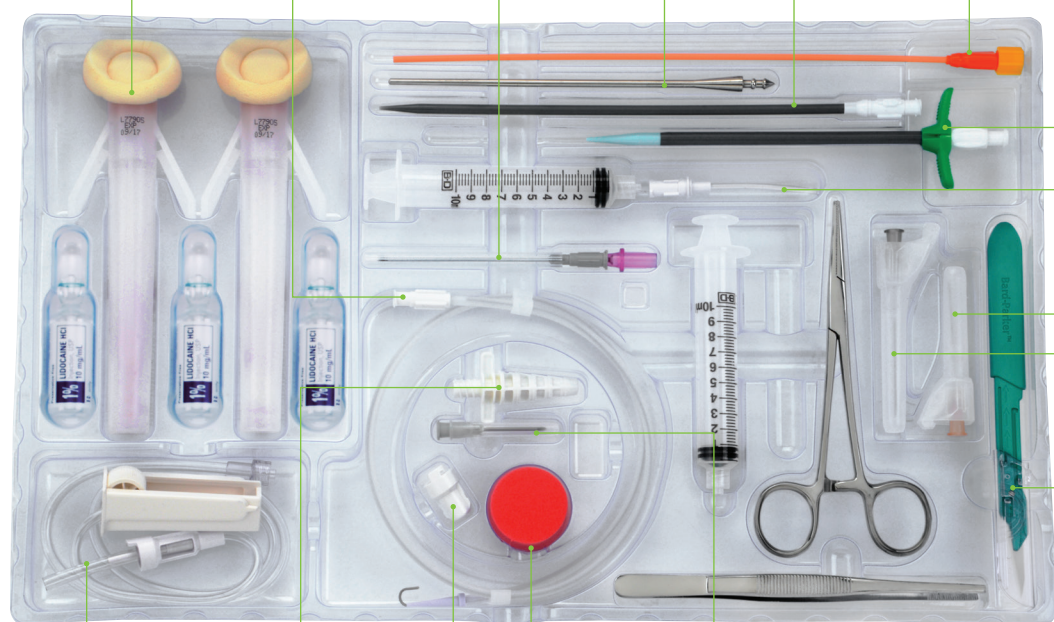

Drainage line with lockable access tip

5-in-1 drainage line adapter (for wall or portable suction)

Valve cap

Needle foam stop

Needle, 17 G

Peel-away introducer, (16 Fr, 13.7 cm dilator length, 10 cm sheath length)

Filter Straw®

22 G and 25 G BD SafetyGlide™ Needles (1 of each)

Safety scalpel

PleurX™ Catheter, 66 cm

Fenestrated drape

Silk suture, 2-0 straight needle

Absorbable suture, 4-0 curved needle

Self-adhesive dressing

Gauze pads

Foam catheter pad

Alcohol pad

Lot number sticker (for traceability)



PleurX™ Pleural Catheter System

Indications for Use: The PleurX™ Pleural Catheter System is indicated for intermittent, long term drainage of symptomatic, recurrent, pleural effusion, including malignant pleural effusion and other recurrent effusions that do not respond to medical management of the underlying disease. The devices are indicated for the palliation of dyspnea due to pleural effusion and providing pleurodesis (resolution of the pleural effusion). The PleurX™ Pleural Catheter is indicated for adults only. The Lockable Drainage Line is used to drain fluid using standard wall suction, water seal drainage system, vacuum bottle, or other appropriate method.

Warnings: Do not put anything except the access tip of the Lockable Drainage Line, Catheter Access Kit, or PleurX™ Vacuum Bottles into the catheter valve since any other device could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

Do not place the PleurX™ Pleural Catheter into the peritoneal space as it could lead to misidentification of the catheter and/or mistreatment of the patient.



PeritX™ Peritoneal Catheter System 50-9000C for ascites management



PeritX™ Peritoneal Catheter System

Indications for Use: The PeritX™ Peritoneal Catheter System is indicated for intermittent, long term drainage of symptomatic, recurrent, malignant and non-malignant ascites that does not respond to medical management of the underlying disease and for the palliation of symptoms related to recurrent ascites. The use of the PeritX™ Peritoneal Catheter for non-malignant ascites is limited to patients who are intolerant or resistant to maximum medical therapy, refractory to large volume paracentesis (LVP) and are not candidates for a trans-jugular intrahepatic portosystemic shunt or LVP. The PeritX™ Peritoneal Catheter is indicated for adults only. The Lockable Drainage Line is used to drain fluid using standard wall suction, water seal drainage system, vacuum bottle, or other appropriate method.

Warnings: Do not put anything except the access tip of the Lockable Drainage Line, Catheter Access Kit, or PleurX™ Vacuum Bottles into the catheter valve since any other device could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

A diagnostic paracentesis should be performed if the patient shows signs or symptoms of possible spontaneous bacterial peritonitis (SBP) such as fever or abdominal pain. If SBP is present, the patient should be treated per institutional guidelines, including systemic antibiotics and repeat diagnostic paracentesis at the end of the antibiotic regimen. For a patient with resolved SBP, the patient should be treated per institutional guidelines, including prophylactic antibiotics to help in the prevention of refractory or recurrent SBP. In the case of refractory or recurrent infections, the catheter should be removed and reinserted at the discretion of the clinician after the SBP has resolved. Do not place the PeritX™ Peritoneal Catheter into the pleural space as it could lead to misidentification of the catheter and/or mistreatment of the patient.

For more information, please contact customer service at **844 823-5433** or visit **bd.com/drainage**

Please consult product labels and package inserts for indications, contraindications, hazards, warnings, cautions, and information for use.

bd.com/drainage BD, Vernon Hills, IL, 60061, U.S.

The BD Logo, ChloraPrep, Hi-Lite Orange, PleurX and PeritX are trademarks of Becton, Dickinson and Company or its affiliates. © 2024 BD. All Rights Reserved. BD-20810v4

